

### **Aquacut End-of-Day Routine**

1. At the end of the day the Aquacut must be purged of all water and powder in its tubing and dosing chambers.
2. To do this power the Aquacut down. Remove the dosing chamber lids. Remove any powder canisters in your Aquacut. Tighten the dosing chamber lids back in place.
3. Remove the water tank and discard any unused water. With the water tank still removed power the Aquacut back up.
4. Fully depress your foot pedal while holding the handle towards a garbage can. Watch the tip of the handle. You should see water and powder coming from the tip of the handle. Keep the foot pedal fully depressed until there is no more water and powder coming from the handle. Finally, switch to the opposite dosing chamber while the foot pedal is still fully depressed, ensuring no water or powder is coming from the tip of the handle again.
5. Power the Aquacut down.
6. Replace the empty water reservoir at this time. Refill the water reservoir with distilled water the next day.

**NOTE: When transporting the Aquacut anywhere, even from counter-to-counter, always remove the powder canisters and water tank before moving the Aquacut. Failing to do so, will result in an over spillage of powder in the dosing chambers, leading to clogged powder lines and a water mess inside your Aquacut.**